

PRODUCT INFORMATION

Keep this guide for future reference. Read all instructions before using the product.

CUSTOMER SUPPORT

Distributed by United Yoram Inc.
Brooklyn, NY 11236
drfranklyns.com
Instagram: @drfranklyns

DOCUMENT CONTROL

Brand: Dr. Franklyn's | Model: ALS-1 | Revision: 1.6 | Issue date: September 2026

SCAN FOR PRODUCT DETAILS



drfranklyns.com

BEFORE FIRST USE

- Inspect the brace, panels, straps and hook-and-loop closures. Do not use if damaged.
- Wear over a thin garment when possible to help protect the skin.
- Adjust the brace while standing comfortably with the spine in a neutral position.
- The brace should feel supportive and snug, never painful or restrictive.

INTENDED USE

The Advanced LSO 631 Back Brace is designed to provide adjustable lumbar support and compression. Use only as directed in this guide or by a healthcare professional.

IMPORTANT

Stop use and seek medical advice if pain increases, numbness develops, skin becomes irritated, or circulation appears impaired.



HEALTH | SUPPORT | RECOVERY



MODEL ALS-1

ADVANCED LSO 631
BACK BRACE

Adjustable lumbar support

READ INSTRUCTIONS COMPLETELY BEFORE USE

1. SIZE AND PREPARE

- 1

OPEN
Lay the support flat with the back exterior facing upward. Open the back cover, unfasten the hook-and-loop closures on each side, and remove both side panels.
- 2

SET THE SIZE
Use the size markings on each panel. Fold each panel to the correct marker.
- 3

REINSTALL
Return the panels to the support, secure them firmly, and close the back cover.

FIT CHECK

- Back panel centered on the spine.
- Lower edge sits comfortably above the hips.
- Waist panel is secure without restricting breathing.

2. APPLY AND ADJUST

- 4

POSITION
Wrap the support around the waist with the back panel centered on the spine. Fasten the waist panel snugly.
- 5


COMPRESS
Take a deep breath and pull both hand loops at the same time until the desired compression is reached. Secure the loops.
- 6

SIDE SUPPORT
Remove or install the lateral panels to decrease or increase side support as needed.
- 7


BACK SUPPORT
Remove or install the back panel to decrease or increase posterior support as needed.

Recheck the fit after sitting, standing or moving. Loosen immediately if the brace causes pressure, tingling or numbness.

VISUAL FITTING GUIDE

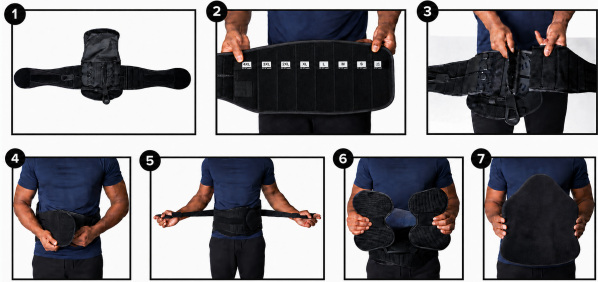

SIZE MARKERS

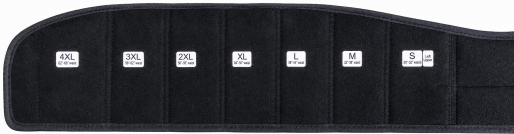

CARE INSTRUCTIONS

- Remove all rigid and removable panels before cleaning.
- Spray with 50% alcohol, wipe clean, and allow to air dry completely.
- Do not machine wash, tumble dry, iron, bleach or expose to direct heat.
- Store flat in a clean, dry place with closures fastened.

WHEN TO CONTACT A PROFESSIONAL

Seek advice before use after recent surgery or injury, or if you have impaired sensation or circulation. Stop use if discomfort persists.